


Urine Monovette® User Guide


1

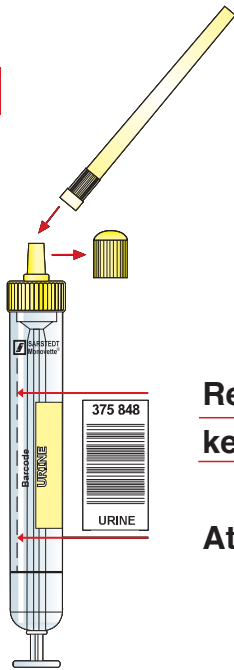

Remove stopper and keep for later use!

Attach plastic straw.

2

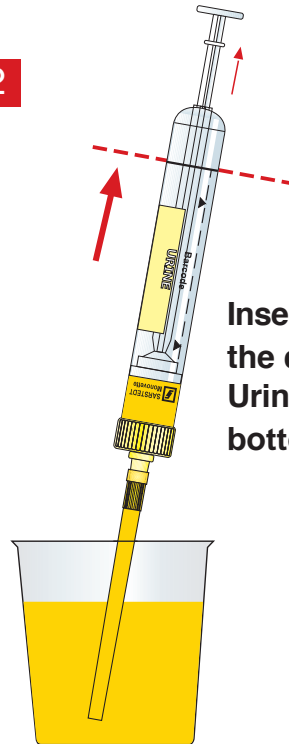


Insert the plastic straw into the container and fill the Urine Monovette® up to the bottom of the label.

3



To empty the plastic tube, hold the Urine Monovette® in an upright position and pull the plunger backwards to the bottom of the tube.

4

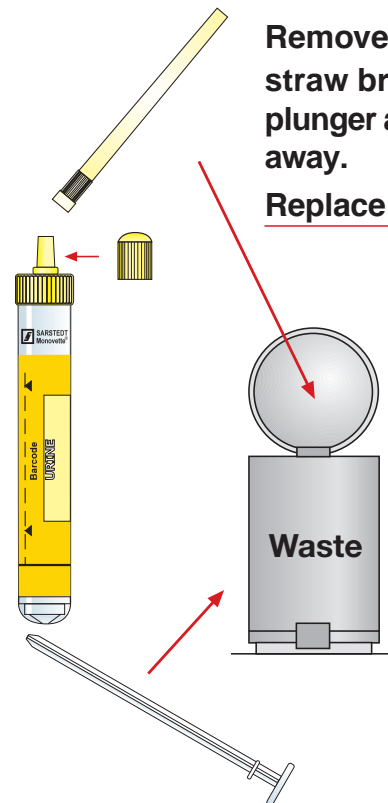

Remove the plastic straw break off the plunger and throw away.

Replace the stopper.

Waste

Technical modifications reserved



SARSTEDT

SARSTEDT Ltd. · 68 Boston Rd, Leicester LE4 1AW · Tel.: 0116 235 9023 · Fax: 0116 236 6099

info@sarstedt.co.uk · www.sarstedt.com